



A new twist on
manually harvesting bone



SafeScraper™ TWIST Cortical Bone Collector



1 Provides 160° cutting area

- Effective on plane, concave and convex bone surfaces

2 Available in curved and straight designs

- Versatile, facilitating access to hard-to-reach posterior regions

3 Large capacity chamber

- Minimizes bone shaving dispersion and retains bone in a sterile chamber to limit patient contamination

4 Lateral opening system

- Provides device stability and may be used as a container for composite graft

5 Viability

- Harvested bone contains viable bone cells and shows high osteogenic potential^{1,2}
- Higher cell viability, cell proliferation, osteogenic potential, and release of growth factors compared to other harvesting methods^{2,3}



SafeScraper TWIST: How it works

1 The bone is collected by pushing the end of the device toward the bone surface with the chamber facing downward and simultaneously pulling the device itself backward.



2 To retrieve collected bone, keep the transparent chamber facing downward, press the lock button sideways, and slide the chamber clockwise.



Each SAFESCRAPER TWIST has a per-patient blade life that effectively harvests up to 5 cc of bone



Ordering Information

Item No.	Description
3598	SAFESCRAPER TWIST Straight (3 units), 3 cc
3987	SAFESCRAPER TWIST Curve (3 units), 2.5 cc

Shelf life of SAFESCRAPER TWIST is three years.

1. Zaffe D, D'Avenia F. A novel bone scraper for intraoral harvesting: a device for filling small bone defects. Clin Oral Impl Res. 2007;18:525-533.
2. Miron RJ, Hedbom E, Saulacic N, Zhang Y, Sculean A, Bosshardt DD, Buser D. Osteogenic potential of autogenous bone grafts harvested with four different surgical techniques. J Dent Res. 2011;90:1428-33.
3. Miron RJ, Gruber R, Hedbom E, Saulacic N, Zhang Y, Sculean A, Bosshardt DD, Buser D. Impact of Bone Harvesting Techniques on Cell Viability and the Release of Growth Factors of Autografts. Clin Impl Dent Rel Res. 2013;15:481-489.

For more information, visit ZimVie.com

ZimVie
4555 Riverside Drive
Palm Beach Gardens, FL 33410
1-800-342-5454
Phone: +1-561-776-6700
Fax: +1-561-776-1272
dentalCS@ZimVie.com

ZimVie
Restoring Daily Life.®

Unless otherwise indicated, as referenced herein, all trademarks and intellectual property rights are the property of ZimVie Inc. or an affiliate; and all products are manufactured by one or more of the dental subsidiaries of ZimVie Inc. (Biomet 3i, LLC, Zimmer Dental, Inc., etc.) and marketed and distributed by ZimVie Dental and its authorized marketing partners. Safescraper is a trademark of C.G.M. S.P.A. Safescraper is manufactured by META Advanced Medical Technology. Contact Zimmer Biomet Dental for references to all claims. For additional product information, please refer to the individual product labeling or instructions for use. Product clearance and availability may be limited to certain countries/regions. This material is intended for clinicians only and does not comprise medical advice or recommendations. Distribution to any other recipient is prohibited. This material may not be copied or reprinted without the express written consent of ZimVie. ZV1670 REV A 09/23 ©2023 ZimVie. All rights reserved.

