



Synergy of
Space and Time

CopiOs Extend[®] Membrane



Space maintaining conformance

- Designed to conform to the defect with enough structural integrity for space maintenance

Extended resorption profile

- Lasts six to nine months,¹ designed to allow implant placement while providing ample time for regeneration

Performs even when primary closure is not attained

- No signs of inflammation or infection in cases where primary closure has not been achieved^{2,3}



Long lasting, resorbable collagen membrane

Performs When Primary Closure Has Not Been Achieved

Clinical experience with CopiOs Extend Membrane has shown that the membrane shows no signs of inflammation or infection in cases where primary closure has not been achieved. The exposed area is healed by soft tissue covering the exposure within a few weeks while the membrane maintains its barrier function.^{2,3}

Indications

CopiOs Extend Membrane is intended for use in oral surgical procedures as a resorbable material for:

- Use in augmentation around implants placed in immediate and delayed extraction socket
- Localized ridge augmentation for later implantation
- Alveolar ridge reconstruction for prosthetic treatment
- Filling of bone defects
- Guided bone regeneration in dehiscence defects
- Guided tissue regeneration procedures in periodontal defects



Take a closer look



Figure A Clinical image showing the extensive bone loss on the palatal area extending past apex.



Figure B CopiOs Extend Membrane was laid over the graft and tacked under the flap.



Figure C Flap sutured over the membrane.



Figure D Two months postoperative.

Clinical photos ©2013 Dr. El Chaar[†] All rights reserved. Individual results may vary.

Ordering Information

Catalog Number	Description
0190Z	CopiOs Extend Membrane, 15 x 20 mm
0191Z	CopiOs Extend Membrane, 20 x 30 mm
0192Z	CopiOs Extend Membrane, 30 x 40 mm

- 1 Data on file with Collagen Matrix, Inc.
- 2 Clinical cases on file with ZimVie Dental.
- 3 Clinical cases on file with Regenity

[†] Dr. El Chaar has or had a financial relationship with ZimVie Dental resulting from speaking engagements, consulting engagements and other retained services.

For more information, visit ZimVie.com

ZimVie
4555 Riverside Drive
Palm Beach Gardens, FL 33410
1-800-342-5454

 **ZimVie**
Restoring Daily Life.®

Unless otherwise indicated, as referenced herein, all trademarks and intellectual property rights are the property of ZimVie Inc. or an affiliate; and all products are manufactured by one or more of the dental subsidiaries of ZimVie Inc. (Biomet 3i, LLC, Zimmer Dental, Inc., etc.) and marketed and distributed by ZimVie Dental and its authorized marketing partners. For additional product information, please refer to the individual product labeling or instructions for use. Product clearance and availability may be limited to certain countries/regions. This material is intended for clinicians only and does not comprise medical advice or recommendations. Distribution to any other recipient is prohibited. This material may not be copied or reprinted without the express written consent of ZimVie. ZV1660 REV A 09/23 ©2023 ZimVie. All rights reserved.

